ALCOOLTEST

SENZA CROMO

IO SONO CONTE

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4) **Autorisation for the activity of OZCM** 5) **Communication sent to the Advanced Institute Health and Minister Productive Activities**

6) **Recording F.D.A. n°3006026304 (USA)** 7) **Experimentation positive du ministère du intérieur Sez. Polizia Stradale n° 300/A/12/33366/144/4/20 del 16.04.2004**

S.O.S. : +39 - 339. 82.82.019

ALCOOLTEST

WITHOUT CHROMIUM

I AM WITH YOU

YOU BREAK ENVELOPE, TO REMOVE THE STOPPERS, TO INSERT THE BALLON AND YOU SWELL IT, TO CONTROL COLOUR AFTER 2 MINUTES (TO THREAD THE VIAL IN THE FISSURE OF THE BALLON UNTIL THE BLOCKING, TO INSERT THE MOUTHPIECE FROM THE OTHER SIDE)

0.3 – 0.4	Euphoric state – Slower Reflexes of one second	Prestations decreased
0.5	Euphoria, reduction in perceptions and balance	Réflexes slowed down
0.8 – 1.2	Personality disorders, balance, reduction in the capacity	Dangerous Driving
1.6 – 2.5	Very reduced perceptions, Hardly sufficient step	Irresponsible Control
3 or more	Strongly mental confusion, until the died or coma	DIED or COMA

Alcooltest Marketing Italia www.alcooltest.net

Via J. F. Kennedy, 8 – 06083 Bastia Umbra (PG) Italie Tél. 339.8282019 Fax 075.8012645

Breath alcohol testing (disposable) CEC 1 or CEC CALLED IN ORIGIN respective ALT3 and ALT4 (also in confection ALTIK)

This alcoholtest DOES NOT CONTAIN CHROMIUM in adaptation to directive 67/548/CEE and 1999/45/CE

Destination of use: disposable analyzer to chemical reaction for survey for indicative alcolemic ratio contained in expired.

Compositions: reagent constituted from knowns them mineral: K, Si, H2SO4 (does not contain cancerogenous substances); balloon in biodegradable material.

The product is classified NOT DANGEROUS – Certification n° 11/2003 of the 15/05/2003 Ecoline spol.s.r.o. to norm Dec.157/98 and 376/2001 Czech Rep. – C.R. < 1

Range of survey of the alcolemic ratio: first change color from the white to the rose from 0.3 mg/l % – Intensity color proportionally to the alcohol amount (tolerance +/- 5 %)

- Alcoholtest fiala CEC 1: 0.2 mg/l % – 0.5 mg/l % (color indicated on external label of the vial)

- Alcoholtest fiala CEC 0.4 mg/l % – 8 mg/l % – 1.2 mg/l %

Expiration and conservation: Produced 18 months before the Expiry Date – To conserve in absence of light and to one not advanced to the +30 °C and not inferior temperature to 0 °C.

Contraindications (When it does not have to be used):

1. Within 15 minutes from the assumption of alcoholic drinks, candies, cakes, etc
2. Within 15 minutes having smoked, clean the teeth or used the oral spray.
3. To conserve in light absence the balloon is the unit of reference for the measurement

Modality and councils of use of vial ALCOOLTEST:

the test is personal and can be used a little of prevention

Use:

1. To remove all and the two stoppers to the extremity of the vial.
2. To thread the vial within the balloon to the indicated extremity the arrow and to put the mouthpiece of rubber in the opposite extremity.
3. To expire (to blow) within the vial, in continuous way so that the balloon is possibly swollen completely. During this phase the vial is heated.
4. To control the result of the test after a minute.

Reading of the Result:

1. To compare the assumed darker coloration from grains situated to you between the two filters to the inside of the vial, with the range indicated on the vial.
2. The coloration of the reagent of the vial, will express the indicative alcolemic ratio with reference to the values prints to you on the external label.

Warnings and precaution for the use:

To avoid the contact with the content of the vial. (S = 26, 30, 45) If it had to be come to contact with the reagent to rinse abundantly with water and to o to the more close Ready Aid.

Smalltment: in binders for riciclabile glass.

This product has been manufactured respecting all the conditions of emergency, but given its characteristic to hold far away from the capacity of the children. The producer and the distributor are not responsible for a use not in compliance with the indications.

Production: Tejas s. r. o.

Distribution: Alcooltest Marketing Italia – Via Kennedy 8 – Bastia U. – PG – Tél. 3398282019, fax 075 8012645, info@alcoholtest.net

Origin certification: The in vial ALCOOLTEST recorded with name original technical "A12/3/4 ALCOHOLTEST DETECTION TUBE" is produced from the company Tejas s. r. o. – Jablonec nad Nisou (Cz)

- 1) Certificate of origin type n° 214-98-0168/D 03879 of which Authorization n° 2/1997 of the 20/6/1997 of the NATIONAL INSTITUTE OF THE SANITATY of Praha (Rep. Ceca) to the Tejas producer s.r.o.
- 2) The test has exceeded the tests in compliance with the directives of the NATIONAL INSTITUTE OF THE PRAGA HEALTH answering to norms ISO 10993 – with certificate ZPP 13-3464/98-1524 the 6/11/98, recorded to the n° 1019;
- 3) Authorization n° 3076 of the Laboratory test with document n° 050099-02 Of the 27/8/2002 of the VVUJ, a.s. of OSTRAVA – RADVANICE (Repubblica Ceca) and identified n° VVUJ-009/H/204 to the n° 49900331 of the 6/2/2004, for report the VVUJ-014977/2004
- 4) Authorization for the activity of OZCM
- 5) Communication send to the Advanced Institute Health and Minister Productive Activities
- 6) Recording F.D.A. n° 3006026304 (USA)
- 7) Positive spermentation of the Ministry of the Interno Sez. Polizia Stradale n° 300/A/2/23366/144/4/20 del 16.04.2004

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